



Specimen Requirements

An mRNA test to help improve patient selection for biopsy.

Patient Clinical Information Requirements:

- Date of birth
- PSA result
- Date of PSA (must be within 6 months of sample collection)
- DRE result
- DRE date
- Family history

Urine Specimen Requirements:

- Urine must be collected immediately following a Digital Rectal Examination (DRE) procedure (first catch). (See SelectMDx Physician Instructions for urine collection and the SelectMDx Urine Collection, Preparation, and Transportation instructions.)
- The urine sample must be transferred into the SelectMDx Urine Preservative Transport Tube immediately after collection. Specimen must be received within 5 days of collection.

Labeling Instructions:

The post-DRE urine sample must be labeled with the patient's:

- First name, last name, date
- Date of birth

Turnaround Time:

5 business days.

Patient Eligibility:

Patient has not been previously diagnosed with prostate cancer.

Cause(s) for Specimen Rejection:

- DRE was not performed prior to obtaining the patient's urine sample.
- Urine sample not received by MDxHealth within 5 days.
- Date of PSA result is more than 6 months prior to urine sample collection.
- Patient had a prostate biopsy within 3 months.

If you have any questions regarding the test ordering or shipping process, do not hesitate to contact MDxHealth Client Services at 866.259.5644 or clientservices@mdxhealth.com.

Jess Savala, Jr., MD, Laboratory Director
CLIA#05D2033858

