



**Step 1:**  
Remove funnel and tube from urine transport kit.




**Step 2:**  
Unscrew tube cap.



**Step 3:**  
Firmly screw funnel onto tube (keep tube upright to prevent spilling the liquid preservative).



**Step 4:**  
Point the spout towards the toilet and begin urinating into the funnel.

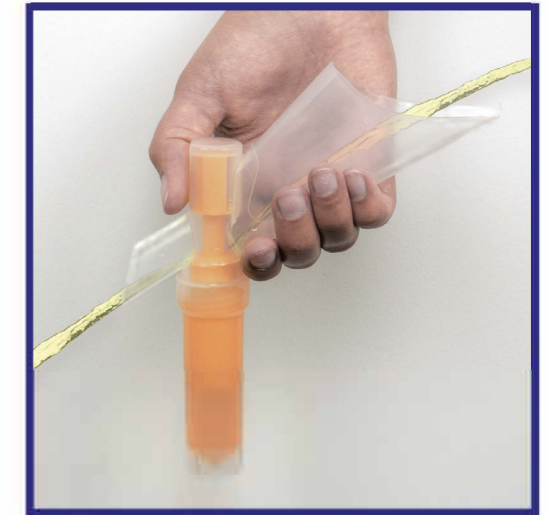

**Step 5:**  
Continue urinating until bladder is emptied.



**Step 6:**  
Unscrew funnel from tube (keep tube upright to prevent spilling the urine sample).



**Step 7:**  
Firmly screw cap onto the tube (keep tube upright to prevent spilling the liquid preservative).



**Step 8:**  
Dispose of the funnel.



**Step 9:**  
Gently invert UrNCollect Tube five times to mix the contents.

**Step 10:**  
Place (1) barcode label on UrNCollect Tube.  
Place (1) barcode label on Test Requisition Form.