# Select MDx<sup>®</sup>

## for Prostate Cancer

### Instructions for Use

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**Patient Eligibility**

Patient has not been previously diagnosed with prostate cancer.

**Physician instructions**

Prior to urine collection, a digital rectal examination (DRE) of the prostate is required. The DRE is necessary to mobilize the prostate cells and exosomes towards the urethra and consequently increase the urinary target mRNA levels for a valid SelectMDx® patient result.

- Conduct a DRE immediately prior to urine collection.
- Apply enough pressure to slightly depress the prostate surface.
- Perform at least three strokes per lobe from the base to the apex and from the lateral to the median line for each lobe as shown in the figure below.

**Requirements of urine sample**

Immediately after a DRE first voided urine must be collected (see Urine Collection and Preparation Instructions). The urine sample must reach the laboratory within 5 days upon collection.

**Requirements for patient data**

These fields are mandatory on the SelectMDx® patient request form:

- Date of Birth
- PSA result
- Date of PSA (within 6 months of urine collection)
- DRE result
- DRE date

**Processing time**

5 working days

**Exclusion criteria**

- Missing DRE prior to urine collection.
- Urine sample reaches the laboratory after 5 days.
- Urine collected from a catheter.
- Serum PSA has been determined > 6 months prior to urine collection.
- Patient has had a prostate biopsy < 2 months prior to urine collection.
- Patient taking medication affecting serum PSA < 6 months prior to urine collection (5α reductase inhibitors such as finasteride and dutasteride and LHRH agonists such as leuporelin).

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### Warnings and Precautions

The SelectMDx for Prostate Cancer Test results should be interpreted in conjunction with other laboratory and clinical data available to the clinician and relevant guidelines in the decision for biopsy. A positive SelectMDx test result suggests an increased risk for prostate cancer detection in a biopsy. However, it is not a definitive indication of the presence of prostate cancer. Following a positive test result, a biopsy is necessary in order to determine whether cancer is in fact present. A very low risk SelectMDx test result does not guarantee the absence of prostate cancer and does not preclude the possibility of developing cancer in the future. Patients with a negative SelectMDx test result should remain under the care and supervision of their urologists with routine screening to mitigate risks associated with delayed cancer detection and treatment. Performance is based on the presence of all relevant data elements; if all data are not available, or 5 $\alpha$ -reductase inhibitors (5-ARIs) have been administered to decrease serum PSA levels, results should be interpreted with caution and AUC of the test may vary.

MDxHealth B.V. | Transistorweg 5 | 6534 AT Nijmegen, Netherlands  
T: +31 (0) 88 327 2727 | F: +31 (0) 88 327 2728 | KvK: 09164747  
E: [clientservices.eu@mdxhealth.com](mailto:clientservices.eu@mdxhealth.com) | [www.mdxhealth.com](http://www.mdxhealth.com)

MDxHealth® and SelectMDx® are registered and pending United States or European Union trademarks of MDxHealth S.A.