

Select mdx is a urine test that helps physicians determine if a patient is at a higher or lower risk for clinically significant prostate cancer prior to biopsy.

### Patient Clinical Information Requirements:

- Date of birth
- Last PSA result
- DRE result
- Prostate volume (only if obtained by MRI or ultrasound)

### Urine Specimen Requirements:

- First catch, post-modified DRE urine

### Labeling Instructions:

- First name and last name
- Date of birth

### Turnaround Time:

- Within 5 days from receipt of specimen<sup>1</sup>

### Patient Eligibility:

- Patient has not been previously diagnosed with prostate cancer

### Causes for Specimen Rejection:

- Modified DRE was not performed prior to obtaining the patient's urine sample
- Urine sample not received by mdxhealth within 5 days
- Patients receiving treatment known to affect PSA levels (eg. 5a-reductase inhibitors such as finasteride and dutasteride)
- Urine collected in an expired collection tube
- Collection date not provided



Xpert One Support is founded on personal attention, responsiveness, and commitment to customer satisfaction. Our Xperts are truly experts on mdxhealth products and processes, from specimen requirements and shipping logistics, to clinical insights. Together with your mdxhealth Molecular Diagnostic Specialist, Xpert One Support strives to make your mdxhealth experience the best it can be.

Physicians, patients, lab technicians, and everyone in the fight against prostate cancer—you can expect more from Xpert One Support.



866-259-5644

(Monday – Friday, 5:00 AM – 5:00 PM PT)



info@mdxhealth.com



Ruben Gamez, MD  
Laboratory Director  
CLIA#05D2033858  
CAP# 8015399

1. When all specimens requirements are met upon receipt.